

# TEACHING FEATURES

1 HEAD TILT



5 COMPRESSIONS



2 JAW THRUST



6 ABDOMINAL THRUSTS



3 CHEST RISE



7 AED TRAINING PADS



4 LANDMARK



## PACKING



 **ACTAR D-fib**  
IT'S A SHOCKER




## IMPORTANT:

**DO NOT PULL** THE BLUE CHESTPLATE OFF FROM THE COLLAR OR YOU WILL DAMAGE THE LATCH.


**TO REMOVE THE CHESTPLATE:**

**PUSH DOWN ON THE ABDOMEN WITH THE HEEL OF ONE HAND AND LIFT THE LOWER EDGE OF THE CHESTPLATE WITH THE OTHER HAND.**

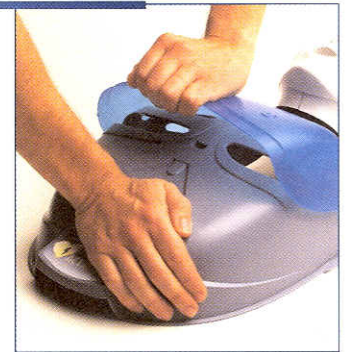


# MANIKIN ASSEMBLY

- 1** SNAP BOTH EAR PINS INTO EACH HOLE AT THE UPPER END OF THE NECK. MAKE SURE THE HEAD IS FACING OUT.

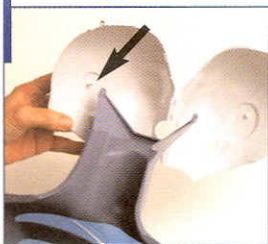

- 2** TO CLOSE THE HEAD: MOVE SLIDER TO THE TOP, SECURE THE UPPER AND LOWER LATCHES, PUSH THE SLIDER FORWARD.

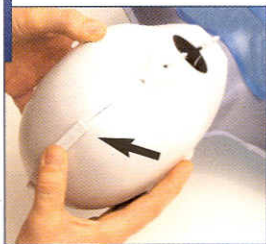

- 3** ATTACH THE MOUTHPIECE TO THE HEAD (INSERT PIN AT TOP AND EACH SIDE OF FACE).



- 4** REMOVE THE CHESTPLATE BY PUSHING DOWN ON THE ABDOMEN WITH THE HEEL OF ONE HAND AND LIFTING THE LOWER EDGE OF THE CHESTPLATE WITH THE OTHER HAND. **WARNING:** DO NOT PULL THE CHESTPLATE OFF FROM THE COLLAR OR YOU MAY DAMAGE THE LATCH.



- 5** ROLL THE LARGE END OF THE LUNG AND FEED IT THROUGH THE MOUTH. OPEN FLAPS OVER THE MOUTH.



- 6** PLACE THE LUNG UNDER THE NECK OF THE CHEST. DO NOT TWIST THE LUNG.



- 7** PULL THE LUNG THROUGH THE LARGE OPENING AT THE BASE OF THE NECK.



- 8** SLIP THE TAB ON THE UNDERSIDE OF THE CHESTPLATE INTO THE SLOT AT BOTTOM OF THE LUNG.



- 9** SNAP THE CHESTPLATE TAB (WITH THE LUNG ATTACHED) INTO THE SLOT AT BOTTOM OF THE CHEST.

